



OPTICAL SERVICE NEUHAUS

Churerstrasse 35 · CH-9470 Buchs · Telefon 081 756 36 44 · Telefax 081 756 55 48

www.optical-neuhaus.com

wneuhaus@rheintal.ch

DERMATOSCOPE E950/4040



500355 + 500383

Camera Setup:

- Rotate power switch to turn camera on. (Power-on lamp will light)
- Press **MODE** button and rotate command dial until **P / S** or **A** is displayed.
- Press the **MF** button and rotate the command dial until the **manual focus** display appears in the monitor. Continue to rotate the command dial until focus display shows **infinity**.
- Select flash mode to **off** position.
- Use the **zoom in / zoom out** button to select proper **magnification / field of view**.

Technical specifications

	Coolpix 5000	Coolpix E4500	Coolpix E950	Coolpix E995	Coolpix E990	Coolpix E990	
Objektiv f mm	7 - 21	8 - 32	8 - 32	8 - 32	8 - 32	8 - 24	8 - 24
Image size	2560x1920	2272x1704	2272x1704	2048x1536	2048x1536	2048x1536	2048x1536
Image quality	fine	fine	fine	fine	fine	fine	fine
Zoom position	tele/21	tele/32	WW/8	tele/32	WW/8	tele/24	WW/8
Object field mm	16	9	30	9	30	12	30
Magn. LCD	2.5X	4X	1.2X	4.0X	1.2X	3.0X	1.2X
Magn.Computer							
at 50%	24X	35X	9X	35X	9X	30X	11X
at 100%	48X	70X	18X	70X	18X	60X	22X