



OPTICAL SERVICE NEUHAUS

Churerstrasse 35 · CH-9470 Buchs · Telefon 081 756 36 44 · Telefax 081 756 55 48

www.optical-neuhaus.com

wneuhaus@rheintal.ch

Dermatoscope E 950/40



500321

Camera Setup:

- turn the **mode** dial to **M Rec**
- turn the **flash off**
- select **manual focus (INF)**
(press the **mode** button while turning the **command** dial)
- use the **zoom in / zoom out** button to select proper magnification / field of view

technical. specifications

	Coolpix 5000	Coolpix E995		Coolpix E990		Coolpix E950	
Objektiv f mm	7 - 21	8 - 32	8 - 32	8 - 24	8 - 24	7 - 21	7 - 21
Image size	2560x1920	2048x1536	2048x1536	2048x1536	2048x1536	1600x1200	1600x1200
Image quality	fine	fine	fine	fine	fine	fine	fine
Zoom position	Tele/21	Tele/32	WA/8	Tele/24	WA/8	Tele/21	WA/7
Object field mm	16	9	30	12	30	12	30
Magn. LCD	2.5X	4X	1.2X	3.0X	1.2X	3.0X	1.2X
Magn.Computer							
at 50%	24X	35X	9X	30X	11X	24X	8X
at 100%	48X	70X	18X	60X	22X	48X	16X